


User Guide Addendum Smart Tip (3x8.5mm Sharp, 27 Gauge)

iovera[®]

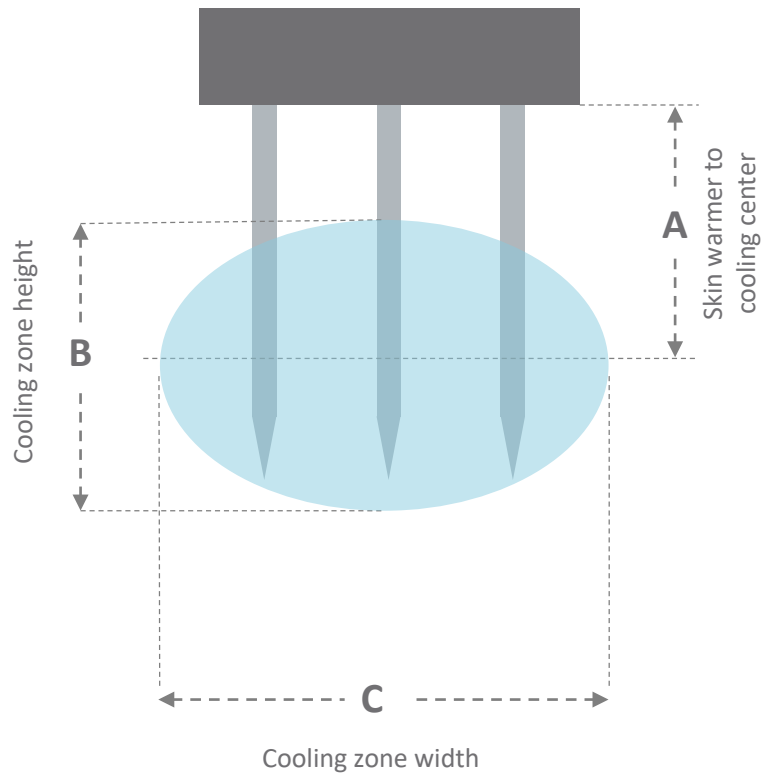

h	A mm (in)	B mm (in)	C mm (in)
STT0309	6.3 (0.25)	6.5 (0.26)	8.0 (0.31)

Dimensions are based on 35-second cooling cycle performed in agarose gel.

This document is provided as a supplement to the User Guide. Always refer to the User Guide for complete operating instructions, warnings and precautions.

Procedure:

- Prep cycle not required for this tip type.
- Insert needle to maximum insertion desired, then perform cooling cycle.



WARNING! To reduce likelihood of skin injury, skin warmer must be pressed against the skin when the needle is inserted before running a treatment cycle.



WARNING! Needle may contact bone in low BMI patients. Patient anatomy should be assessed prior to tip selection.